






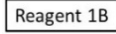

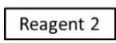

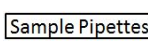

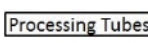

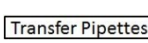

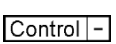

O. REFERENCES

- Hillyer et al. Bacterial Contamination of Blood Components; Risks, Strategies, and Regulation. Joint ASH and AABB Educational Session in Transfusion Medicine. Hematology 2003; 575-589
- AABB. Bulletin 03-12: Further Guidance on Methods to Detect Bacterial Contamination of Platelet Components. AABB, Bethesda, MD, 2003.
- Silva M, editor. Standards for Blood Banks and Transfusion Services. 24th ed.: AABB, Bethesda, Maryland, 2006.
- Larsen CP, Ezligini F, Hermansen NO, Kjeldsen-Kragh J. Six years' experience of using the BacT/ALERT system to screen all platelet concentrates, and additional testing of outdated platelet concentrates to estimate the frequency of false-negative results. Vox Sang 2005;88(2):93-7.
- Blajchman MA, Beckers EA, Dickmeiss E, et al. Bacterial detection of platelets: current problems and possible resolutions. Transfus Med Rev 2005;19(4):259-72.
- Murphy WG, Foley M, Doherty C, et al. Screening platelet concentrates for bacterial contamination: low numbers of bacteria and slow growth in contaminated units mandate an alternative approach to product safety. Vox Sang 2008;95(1):13-9.
- Pearce S, Rowe GP, Field SP. Screening of platelets for bacterial contamination at the Welsh Blood Service. Transfus Med 2011;21(1):25-32.
- Dumont LJ, Kleinman S, Murphy JR, et al. Screening of single-donor apheresis platelets for bacterial contamination: the PASSPORT study results. Transfusion 2010;50(3):589-99.
- Benjamin, R. and Wagner, S. The Residual Risk of Sepsis: Modeling the effect of concentration on bacterial detection in 2-bottle culture systems and estimation of false-negative culture rates. Transfusion 2007; 47:1381-1389
- Jacobs MR et al. Relationship between bacterial load, species virulence, and transfusion reaction with transfusion of bacterially contaminated platelets. Clin Infect Dis 2008;46(8):1214-20
- Jacobs MR et al. Detection of bacterial contamination in prestorage culture-negative apheresis platelets on day of issue with the Pan Genera Detection test. Transfusion 2011; 51:2573-2582
- Fischer. Physiology of Lipoteichoic Acids in Bacteria. Advances in Microbial Physiology 1988; Vol. 29; 233-301
- Rietschel et al. Bacterial Endotoxin: Chemical Constitution, Biological Recognition, Host Response, and Immunological Detoxification. Current Topics in Microbiology and Immunology 1996; 216:39-81
- Jackson et al. Monoclonal Antibodies to Immunodeterminants of Lipoteichoic Acids. Infection and Immunity 1984. Vol 43. No. 3: 800-803
- Lugowski et al. Anti-endotoxin antibodies directed against *Escherichia coli* R-1 oligosaccharide core-tetanus toxoid conjugate bind to smooth, live bacteria and smooth lipopolysaccharides and attenuate their tumor necrosis factor stimulating activity. FEMS Immunology and Medical Microbiology 1996; 16:31-38
- Kuehnert et al. Transfusion-transmitted bacterial infection in the United States, 1998 through 2000, Transfusion 2001; 41:1493-1499
- Yomtovian, R. Bacterial contamination of blood: lessons from the past and road map for the future, Transfusion 2004; 44: 450-460
- 29CFR 1910.1030; Occupational Safety and Health Standards: Bloodborne Pathogens
- 29CFR 1910.1200; Occupational Safety and Health Standards: Hazard Communication
- ST/SG/AC.10/30/Rev.4 Globally Harmonized System of Classification and Labelling of Chemicals (GHS)
- World Health Organization Laboratory Biosafety Manual. Geneva: World Health Organization; 2004
- Greenberg J et al. Influence of Lipoteichoic Acid Structure on recognition by the Macrophage Scavenger Receptor. Infect Immun 1996;64(8):3318-25
- Hogg S et al. Occurrence of Lipoteichoic Acid in Oral Streptococci. Int J Syst Bacteriol 1997;47(1):62-6

ProClin 300 is a registered trademark of Rohm and Haas Company.

Pan Genera, Pan Genera Detection, PGD, 3O D W H O H W PGD*prime* and Verax Biomedical are registered trademarks of Verax Biomedical Incorporated. © 2020 Verax Biomedical Incorporated

Keys to Symbols Used

	List Number		Platelet PGD <i>prime</i> Test Device
	Lot Number		Reagent 1A
	Expiration Date		Reagent 1B
	Temperature limitations		Reagent 2
	Manufacturer		Sample Pipettes
	Consult instructions for use		Processing Tubes
	Attention, see instructions for use		Transfer Pipettes
	Do not re-use		Negative Control
			Positive Control



VERAX BIOMEDICAL

148 Bartlett Street
Marlborough, MA 01752 USA

For Technical Support
Verax Biomedical Technical Support
1.866.948.3729

