

Platelet PGDprime® Processing Tubes, Sample and Transfer Pipettes



A. INTENDED USE

The Platelet PGDprime Processing Tubes, Sample and Transfer Pipettes are only for use with the Platelet PGDprime Test.

B. SUMMARY AND EXPLANATION OF THE TEST

Platelet PGDprime Processing Tubes, Sample and Transfer Pipettes are single use, disposable accessories for use with the Platelet PGDprime Test. They are used in the processing of platelet samples prior to addition to the Platelet PGDprime Test Device and in the processing of Platelet PGDprime Controls.

C. PRINCIPLES OF THE PROCEDURE

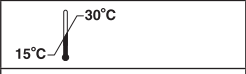
Platelet PGDprime Processing Tubes, Sample and Transfer Pipettes are specifically for use with the Platelet PGDprime Test. Similar laboratory disposables should not be substituted.

D. REAGENTS AND MATERIALS

Materials Provided

Processing Tubes, Sample and Transfer Pipettes for the Platelet PGDprime Test **REF** PRM100D-CE

Includes the following:

	100 Tests
Processing Tubes	100 each
Sample Pipettes	100 each
Transfer Pipettes	100 each

Materials Required

Platelet PGDprime Test

REF PRM100-CE 100 Tests

Materials Required But Not Provided - Refer to the Platelet PGDprime Test Package Insert.

E. WARNINGS AND PRECAUTIONS

For *In Vitro* Diagnostic Use

Warnings

1. Read the Platelet PGDprime Test and Platelet PGDprime Controls package inserts completely before using the product. Follow the instructions carefully. Not doing so may result in inaccurate test results.
2. The Platelet PGDprime Processing Tubes, Sample and Transfer Pipettes have been validated for use with the Platelet PGDprime Test only. Do not substitute similar laboratory supplies.

Handling Precautions

Handle and perform test properly: Do not re-use Processing Tubes or Pipettes.

F. STORAGE INSTRUCTIONS

Store Platelet PGDprime Processing Tubes, Sample and Transfer Pipettes at 15 - 30 °C.

For the following categories - Refer to the Platelet PGDprime Test Package Insert and the Platelet PGDprime Controls Package Insert.

G. SPECIMEN COLLECTION AND PREPARATION FOR ANALYSIS

H. TEST PROCEDURE




I. LIMITATIONS

J. REFERENCES

Platelet PGDprime and Verax Biomedical are registered trademarks of Verax Biomedical Incorporated.

© 2021 Verax Biomedical Incorporated

Keys to Symbols Used

IVD	<i>In vitro</i> diagnostic medical device
LOT	Lot Number
REF	List Number
Sample Pipettes	Sample Pipettes
Transfer Pipettes	Transfer Pipettes
Processing Tubes	Processing Tubes
	Temperature limitation
	Manufacturer
	Consult instructions for use
EC REP	Authorized Representative

 **VERAX BIOMEDICAL**
148 Bartlett Street
Marlborough, MA 01752 USA



EC REP **EMERGO EUROPE**
Prinsessegracht 20
2514 AP The Hague
The Netherlands